

OptiFlex[®]T

Knee CPM Trolley



ISO 13485 Certified Company

DJO, LLC
1430 Decision St
Vista, CA 92081 USA
Phone: 1-800-592-7329 USA
Phone: 1-317-406-2209
Fax: 1-317-406-2014

chattgroup.com
© 2010 DJO, LLC


CE
0473

45255D

Setup



1. Using the 3/16" hex wrench (supplied), completely loosen the clamp hex screw.

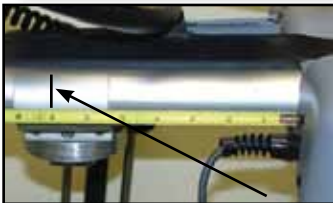

2. Measure 7" (17.8 cm) from the motor housing of the OptiFlex3 unit and place unit into clamp with the 7" (17.8 cm) mark in center of clamp.



NOTE: It may be necessary to obtain another person's help in mounting the OptiFlex3 unit to the OptiFlex T.

3. Using the 3/16" hex wrench, tighten the clamp securely.

The precautionary instructions found in this guide are indicated by specific symbols. Understand these symbols and their definitions before operating this equipment. The definition of these symbols are as follows:



=CAUTION- Text with a "CAUTION" indicator will explain possible safety infractions that could have the potential to cause minor to moderate injury or damage to equipment.



=WARNING- Text with a "WARNING" indicator will explain possible safety infractions that will potentially cause serious injury and equipment damage.



=DANGER- Text with a "DANGER" indicator will explain possible safety infractions that are imminently hazardous situations that would result in death or serious injury.



=REFER TO INSTRUCTION MANUAL/BOOKLET



=PINCH POINT

CE
0473

CAUTION

- Support the unit with both hands when making any adjustments.
- DO NOT exceed trolley weight capacity of 40 lbs (18.12 kg).
- Keep Quick Start instruction card with unit at all times.
- Read, understand, and practice the precautionary and operating instructions found in the manual that was shipped with the OptiFlex3. Know the limitations and hazards associated with the OptiFlex3. Observe any and all precautionary and operational decals placed on the unit.
- Regularly monitor the proper operation of the trolley, making sure to note any worn or wearing parts. If the functional degradation of the trolley is apparent, contact your DJO, LLC dealer or Customer Service Department for assistance.

WARNING

- Never place your hands or feet near the working mechanism of the unit when making any and all adjustments to unit.
- Do not sit or stand or allow patients to sit or stand on the trolley.
- Always place the OptiFlex Trolley in the storage position when not in use.
- The OptiFlex Trolley is for use only with the OptiFlex3 unit. Do not use accessories manufactured by other companies on this trolley. DJO, LLC is not responsible for any consequence resulting from using products manufactured by other companies. The use of other accessories or cables may result in increased emissions or decreased immunity of this trolley and safety of the patient.
- Use of other parts or materials can degrade minimum safety.
- Keep hair, loose clothing, loose bedding, fingers and toes away from the hinge components and pinch points of the unit.
- Do not use the trolley outdoors, on wet or gel filled surfaces. Use only on firm, flat, stable level surfaces to ensure stability of the trolley while in operation.
- When pulling out the T handle, make certain you also have a grasp on the Swivel Release Knob. The Swivel Release Knob can swing up and cause injury or damage other equipment.
- Make certain the Casters are locked before operating the Trolley.

DANGER

- Do not allow anything to be under the unit while the trolley is descending.
- The trolley must be resting on the patient bed to ensure the stability of the OptiFlex3 unit.
- Lock hospital bed while in use.
- When the OptiFlex3 unit is not attached to the trolley, use caution when stepping on the Height Adjustment Pedal. The upward force could cause injury.

Positioning for Storage and Transport



1. While holding onto the Swivel Release Knob, pull out the T Handle.

NOTE: The OptiFlex3 unit must be properly positioned in line with the Clamp Support Assembly before releasing the T Handle.

WARNING

When pulling out the T Handle, make certain you also have a grasp on the Swivel Release Knob. The Swivel Release Knob can swing up and cause injury or damage other equipment.



2. With T Handle pulled, carefully place the OptiFlex3 unit in the vertical position. Release T Handle.
3. While stepping on the height adjustment pedal, press down on the Clamp Support Assembly to completely lower the unit.

NOTE: Always store and transport the unit as shown.



Positioning for Patient Use



1. To place unit in operational position, simply tilt the unit to the horizontal position until the T Handle engages the Clamp Support Assembly.
2. Loosen the Swivel Release Knob by turning counterclockwise.



3. Rotate the unit 90° as shown.
4. Tighten the Swivel Release Knob.

NOTE: The unit may be rotated in either direction or up to approximately 300° as desired.



5. Step on the Height Adjustment Pedal and pull up on the Clamp Support Assembly to raise the unit. When necessary, step on the Height Adjustment Pedal while pressing down on the unit to lower the unit.

Positioning for Patient Use *(continued)*



6. Place the unit at the bedside as shown. Make certain it is comfortably positioned for patient limb placement in the OptiFlex3 unit.



7. Step on the black plastic Caster Tab to prevent the trolley from moving while you are making adjustments on the patient's bed.



8. Once desired position is obtained, make sure the OptiFlex3 unit is resting level on the bed.

Warranty Information

DJO, LLC ("Company") warrants that the OptiFlex3 T Knee CPM Trolley ("Product") is free of defects in material and workmanship. This warranty shall remain in effect for one year (12 months) from the date of original consumer purchase. If this Product fails to function during the one year warranty period due to a defect in material or workmanship, at the Company's option, the Company or the selling dealer will repair or replace this Product without charge within a period of thirty (30) days from the date on which the Product is returned to the Company or the dealer. All repairs to the Product must be performed by a service center certified by the Company. Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

To participate in warranty coverage, this Product's warranty registration card (included with Product) must be filled out and returned to the Company by the original owner within ten business days of purchase.

This Warranty Does Not Cover:

- Replacement parts or labor furnished by anyone other than the Company, the selling dealer or a certified Company service technician.
- Defects or damage caused by labor furnished by someone other than Company, the selling dealer or a certified Company service technician.
- Any malfunction or failure in the Product caused by product misuse, including, but not limited to, the failure to provide reasonable and required maintenance or any use that is inconsistent with the Product User's Manual.

COMPANY SHALL NOT BE LIABLE IN ANY EVENT FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES.

Some locations do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

To obtain service from company or the selling dealer under this warranty, a written claim must be made within the warranty period to the Company or the selling dealer. Written claims made to the Company should be sent to:

DJO, LLC
1430 Decision Street
Vista, CA 92081 USA
T: 1-800-592-7329 USA
T: 1-317-406-2209
F: 1-317-406-2014
and

This warranty gives you specific legal rights and you may also have other rights which vary from location to location.

The Company does not authorize any person or representative to create for it any other obligation or liability in connection with the sale of the Product.

Any representative or agreement not contained in the warranty shall be void and of no effect.

THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.